



#### FORMLABS APPLICATION GUIDE:

## Dental Casting and Pressing from 3D Printed Patterns

Dental milling centers and laboratories have been casting and pressing metal and ceramic parts for over ten years, but existing CNC and 3D printing solutions are often costly and difficult to use. This application guide details the simplified process of casting and pressing dental crowns, bridges, and frames using the Form 2 desktop stereolithography 3D printer and Castable Wax Resin, from start to finish.

Tested at length by dental technicians, Formlabs Castable Wax Resin provides accurate, sealed margins and contains 20% wax for reliable casting with clean burnout. Printed patterns are strong enough to handle with no post-cure required, allowing for a faster, simpler workflow.

## Table of contents

1. Scan .....	3
2. Design.....	3
3. Print .....	5
4. Prepare .....	9
5. Investment .....	11
6. Burnout Process .....	12
7. Casting or Pressing Process.....	14

## Essentials

### **Made by Formlabs**

[Form 2 3D printer](#)

[Castable Wax Resin](#)

[PreForm print preparation software](#)

[Form Wash](#) or [Finish Kit](#)

### **Made by Third Parties**

Digital impression

3D intraoral scanner

Desktop model scanner

Dental CAD software

Dental burnout oven

Investment ring or form

Investment material

Casting machine

Metal or ceramic ingots

Dental lab high speed tools  
for finishing

## 1. Scan

Dental CAD software requires a digital impression to create a crown, bridge, or frame. Capture the digital impression either directly, using an intraoral scanner, or indirectly, using a desktop optical scanner, to scan a polyvinyl siloxane (PVS) impression or stone model. Unless working inside a complete scan and CAD system, export the scan as an open file in STL format.



## 2. Design

After capturing the digital impression, import the scan files into the dental CAD software and design the restoration or appliance. Select dental CAD software that offers open STL file export to ensure compatibility with Formlabs PreForm print preparation software.

Exact importing and design procedures vary by software package. For additional help, contact the software reseller or manufacturer.




### 2.1.1 Crown and Bridge Design

Design your fixed case using the material settings provided by Formlabs (see “CAD Material Settings”).




### 2.1.2 Removable Partial Denture Frame Design

To increase consistency and reduce distortion of 3D printed frame patterns, add a stabilization bar with a diameter of 2.5 mm.

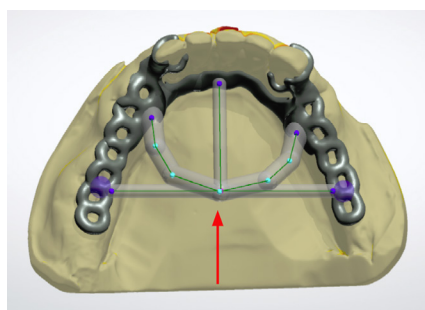

When designing clasps thinner than 0.8 mm, connect the clip arms together for the most accurate results.

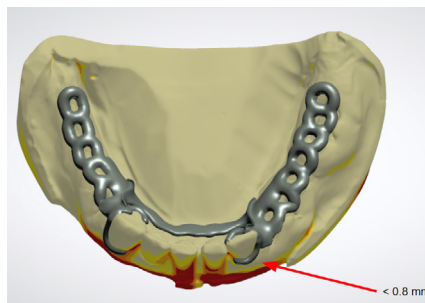

**Tip:** You can create the casting sprues in the CAD software and print them directly. This saves time that would be spent manually placing sprues.

### CAD Material Settings

	CROWN AND BRIDGE	REMOVABLE PARTIAL DENTURE
Minimum Thickness	0.5 mm	0.5 mm
Drill Compensation	Disabled	Disabled
Cement Gap and Extra Cement Gap	0.03 mm and 0.10 mm	-
Margin Offset	0.15 mm	-

### MATERIALS FILES AND LIBRARY

3Shape	<a href="#">Download DME preset file &gt;</a>
--------	-----------------------------------------------



## 3. Print

### 3.1 Import Crown, Substructure, or Frame Files into PreForm

Open Formlabs PreForm print preparation software and import the STL or OBJ file(s) generated by the dental CAD software.

### 3.2 Configure Print Settings


In the “Print Setup” menu in PreForm, select your printer based on its Serial Name. Select “Castable Wax” from the “Material” menu. Print wax patterns at 0.05 mm thicknesses to reduce print times and ensure clinically acceptable accuracy.

### 3.3 Orient Parts for Printing With Supports


Always print with supports. For the best results and the lowest post processing effort, follow the best practices below.

### 3.3.1 Crowns and Bridges

Position crown and bridge patterns with the fitting areas or margins facing upwards. This prevents the possibility of adding supports to the most critical part of the restoration.

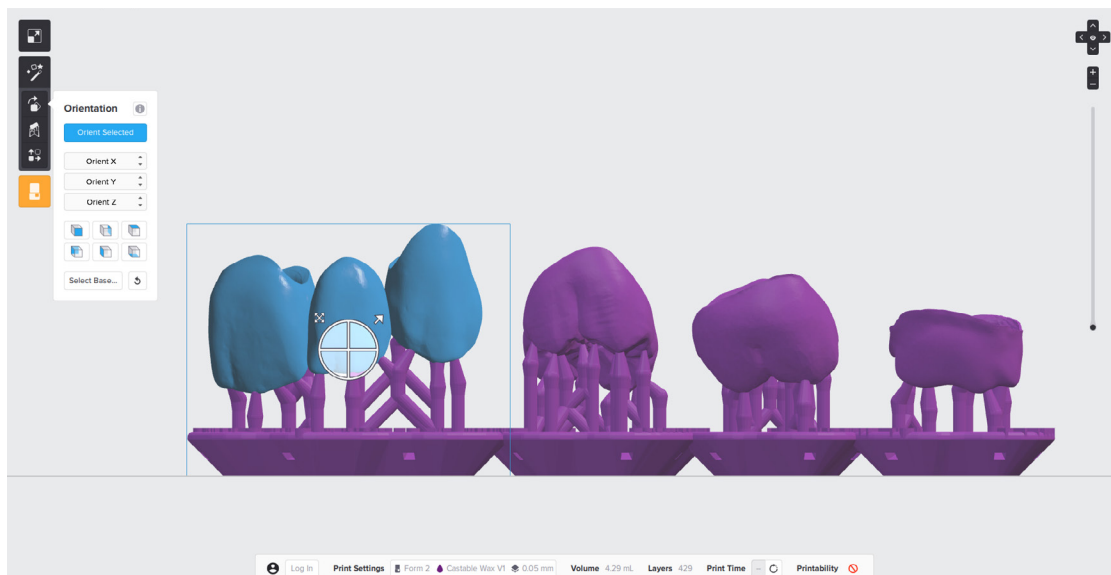


For full contour crown and bridge patterns, place support touchpoints on the cusp tips and ideally not on the contact areas of the crown.

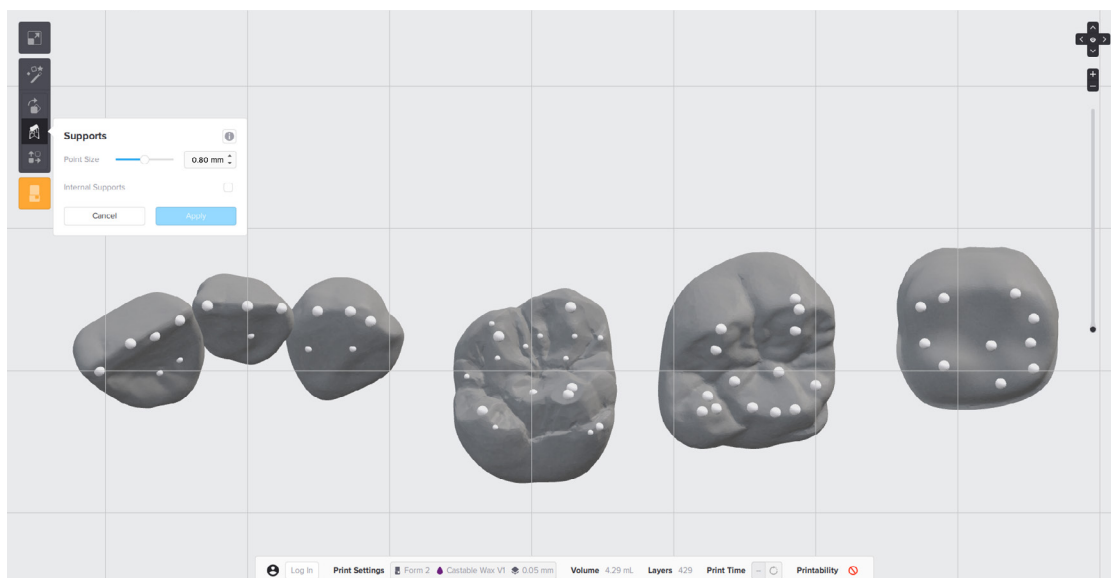




Always check the margin and internal aspect of the restoration to confirm that supports have not been generated in those critical areas. If additional support is needed, manually add supports where appropriate.

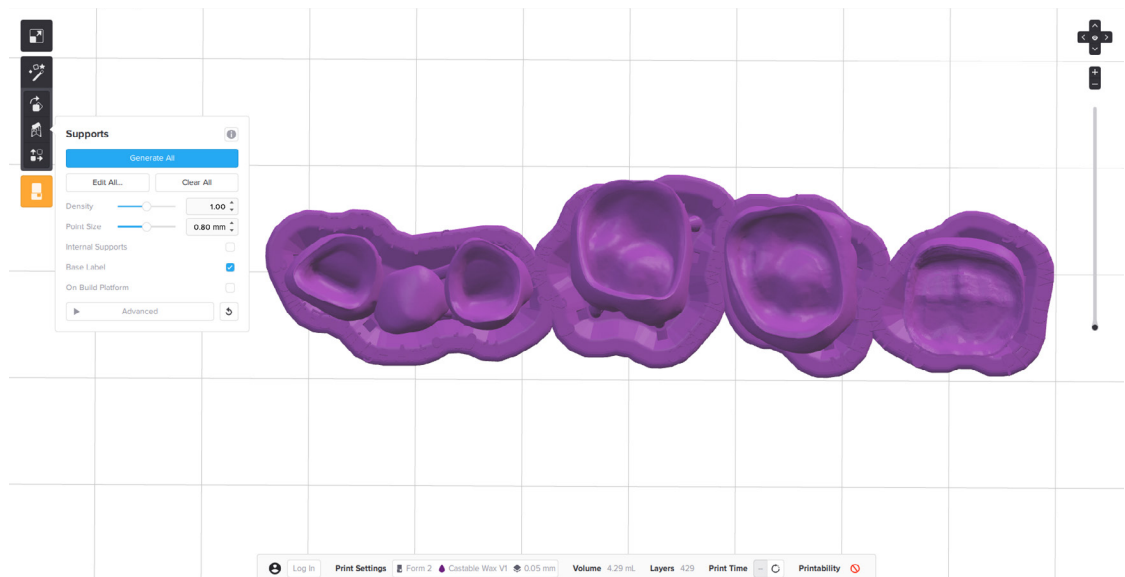


### 3.3.2 Removable Partial Denture Frame


Position your RPD frame with the intaglio surfaces facing up. Manually remove and place supports as needed by clicking “Edit All” in the “Supports” menu. Place supports on the outer surfaces of the clasps.

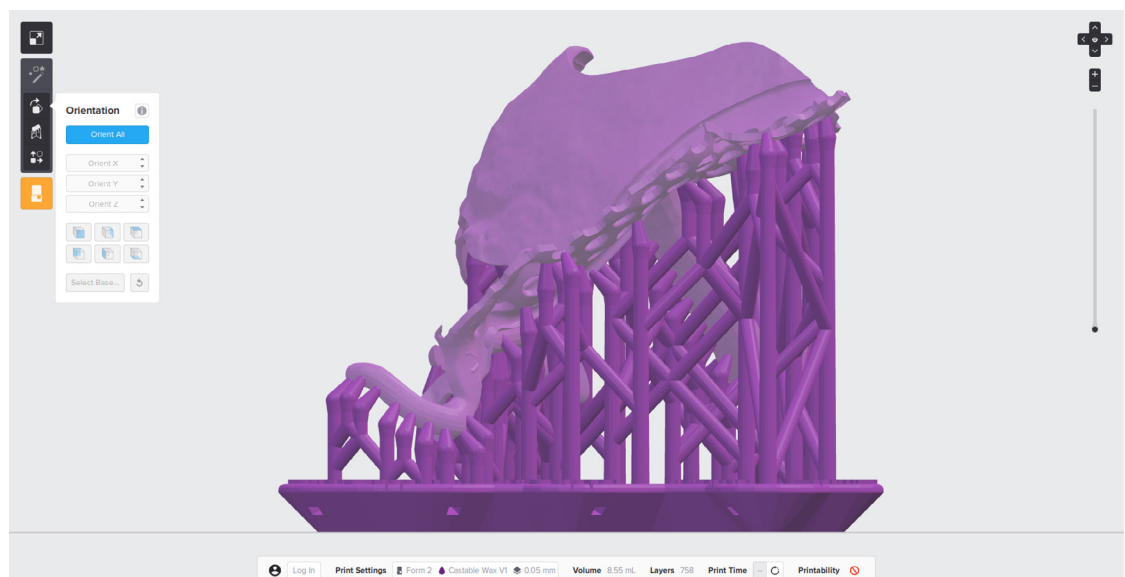

Avoid placing supports on surfaces that will come in contact with the patient's dentition or other important areas of the partial denture frame.

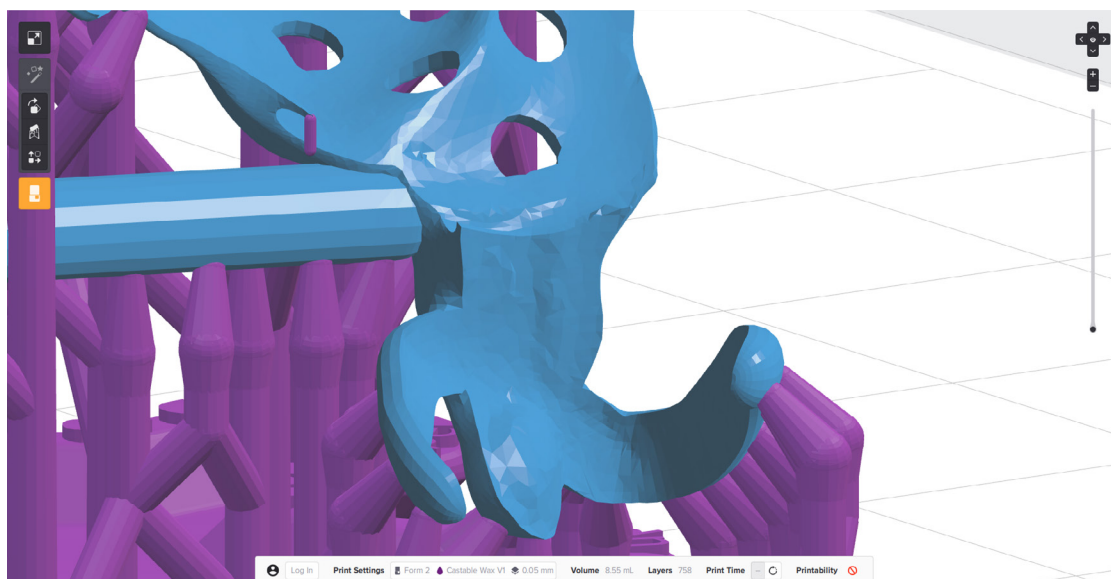

Support each clasp with 4-5 evenly distributed supports along the clasp tip to the shoulder.

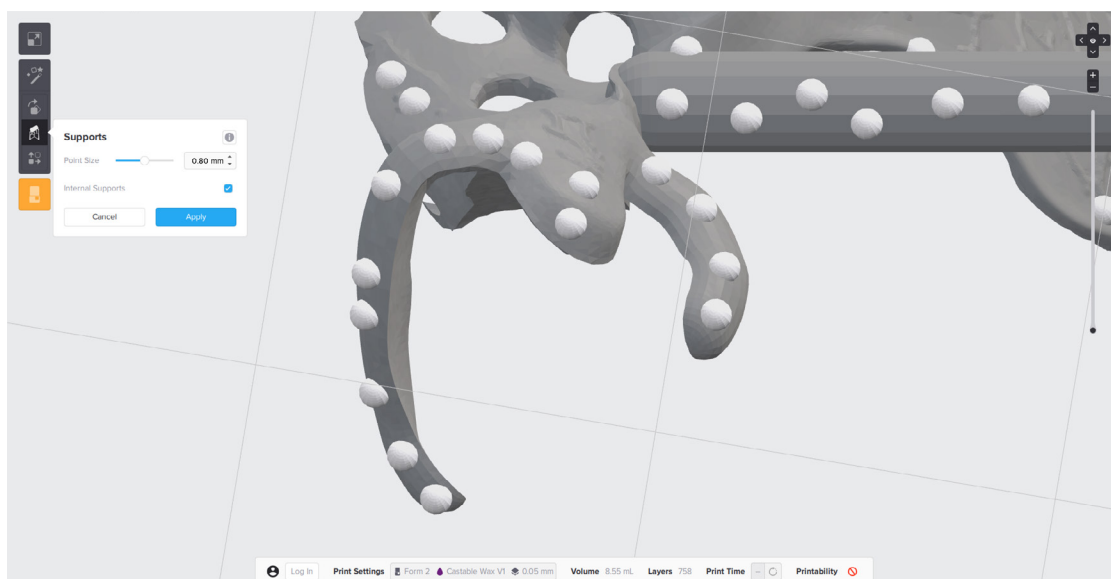

### 3.4 Prepare the Printer and Resin

Thoroughly agitate the resin cartridge by shaking and rotating it several times. Insert the resin tank, the Castable Wax Resin cartridge, and the build platform into the printer.

### 3.5 Print

Click the orange “Print” button in PreForm, then upload the job to the printer. Once the print is uploaded, press the button on the printer to start the print.





## 4. Prepare

### 4.1 Wash Parts

If you are using a Form Wash automated washing system, leave the printed parts on the build plate for washing. Align the top lip of the build platform with the arms of Form Wash's platform mount, and fully insert to touch the back. Wash parts with isopropyl alcohol (IPA) with a concentration of 90% or higher (99% is recommended). Set Form Wash to 15 minutes, and start the wash cycle.

If you are using a Finish Kit, carefully remove parts from the build platform once the build is complete. Firmly insert the included scraper into the chamfer of the support raft, then twist the scraper gently from side to side. Rinse the parts in two buckets of isopropyl alcohol, with an initial bath of 10 minutes, and a second bath of 5 minutes. IPA with a concentration 99% or higher is recommended but 90% or higher will work.



**Notice:** Washing Castable Wax parts will stain the IPA and future washed parts purple. To avoid staining parts printed with other resins, wash Castable Wax parts in a dedicated Form Wash or Finish Kit.

Air dry parts or use a compressed air hose to blow IPA away from surfaces. Inspect parts closely to ensure all uncured resin has been removed. Repeat wash if necessary. Let the patterns dry completely before use.

#### No Post-Cure Required

Castable Wax Resin was designed for use without post-curing. For the highest accuracy, we do not recommend post-curing printed patterns, as this can result in a small amount of shrinkage. Print, process, and cast or press patterns as soon as possible after printing. If this is not possible, store washed Castable Wax patterns in a cool and dark place.

#### 4.2 Support Removal

Use the flush cutters included in the Finish Kit to carefully cut the supports at the touchpoints (where they attach to the part). Consider wearing safety glasses when cutting the supports, as the material may be brittle and small pieces of supports can fly off. Supports can also be removed using dental handpieces, such as cutting discs, burs, and polishing wheels.

**Notice:** Very thinly designed parts, like removable partial dentures (RPDs), are easy to break during the support removal process. Using a cutting disc is recommended.

## 5. Investment

### 5.1 Best Practices for Setting Up an Investment Mold

#### Crowns and Bridges:

Position the crowns and bridges in the mold outside the heat center, about 5 mm from the mold wall. Individual crowns can be grown directly onto the casting mold former using a 2.5 mm wax wire (optionally with a cast reservoir). Bridges, on the other hand, should be connected to a 5 mm distribution channel with a 2.5 mm wax wire at a 45 ° angle and waxed to the casting mold shaper using a 4 mm wax wire.



#### Cast Partial Dentures:

Always fix the sprues to the most solid areas of the structure. Two curved 3.5 to 4 mm casting channels are recommended, connected to the funnel 5 to 10 mm above the highest point of the object. Thicker areas of the construction only accessed via thin areas can be optionally connected to an additional 3 mm casting channel, as the alloy will always flow from thick to thin.



#### Pressable Ceramics:

Consider the following when designing the sprue channels of pressed ceramic patterns:

- Crowns and bridges are sprued directly with a 2.5 to 3 mm wax wire (without corners and edges) at a 45 to 60 ° angle to the mold cavity former.
- Place sprues on the thickest area of the pattern. For crowns, this is generally a cusp tip. Bridges should have a sprue on each abutment and pontic (do not attach a sprue to a bridge connector).
- The casting channel should be between 4 and 8 mm long, and the total length including the press object should not exceed 16 mm long.
- The distance to the wall of the silicone ring should be at least 10 mm.





## 6. Burnout Process

Casting with Castable Wax Resin requires a high heat bonded phosphate investment. Formlabs tested burnout schedules with two investments: BEGO Bellavest SH and WiroFine. The burnout schedules are the same for both investments with variance on the final temperature. Always refer to the investment manufacturer's recommendations.

INVESTMENT	IDEAL FOR	MANUFACTURER INFORMATION
BEGO Bellavest SH	Pressing and casting crowns and bridges	<a href="#">Product details &gt;</a> <a href="#">Manufacturer guide &gt;</a>
Bego WiroFine	RPD frame casting	<a href="#">Product details &gt;</a> <a href="#">Manufacturer guide &gt;</a>

## 6.1 Standard Burnout Schedule

DESCRIPTION	PHASE	TIME TO TEMP	SCHEDULE °C	SCHEDULE °F
Bench Setting	Hold	30 mins	Room Temp	
Insert Mold	Ramp	50 mins	5 °C/min	41 °F/min
Drying	Hold	30 mins	250 °C	482 °F
Thermal Transition	Ramp	45 mins	7 °C/min	44.6 °F/min
	Hold	30 mins	570 °C	1058 °F
Burnout	Ramp	-	7 °C/min	44 °F/min
	Hold	60 mins	Alloy Final Temp	
		Precious metal	700 °C	1292 °F
		Precious metal-to-ceramic alloys	850 °C	1562 °F
		Non-precious metal	900 °C	1652 °F
		Pressable ceramics	Up to 900 °C	Up to 1652 °F
		Non-precious RPD frame	950 to 1050 °C	1742 to 1922 °F

## 6.2 Short Burnout Schedule

DESCRIPTION	PHASE	TIME TO TEMP	SCHEDULE °C	SCHEDULE °F
Preheat Oven		-	900 °C	1652 °F
Bench Setting	Hold	20 mins	Room Temp	
Insert Mold	Hold	20- 30 mins	900 °C	1652 °F
Burnout	Ramp	-	7 °C/min	44.6 °F/min
	Hold	60 mins	Alloy Final Temp	-
		Precious metal	700 °C	1292 °F
		Precious metal-to-ceramic alloys	850 °C	1562 °F
		Non-precious metal	900 °C	1652 °F
		Pressable ceramics	Up to 900 °C	Up to 1652 °F
		Non-precious RPD frame	950 to 1050 °C	1742 to 1922 °F

## 7. Casting or Pressing Process

For a successful casting or pressing process, observe the manufacturer's specifications, especially matching the preheating and casting or pressing temperature to the alloy.



**Notice:** Castable Wax is 21% heavier than standard wax. To figure out the optimal amount of material to press or cast into your pattern, use the following formula:

**(Standard wax weight) x 1.21 = Castable Wax Max Weight**

**For example, if your alloy or ceramic recommends 1 ingot per .75 g in standard wax, the correct weight per 1 ingot of Castable Wax would be .91 g.**

Alloy Manufacturers

[Bego >](#)

[Argen >](#)

[Dentaurum >](#)

## Additional Resources

[Technical Data Sheet: Castable Wax Resin >](#)

Explore [Formlabs dental resources](#) for in-depth guides, step-by-step tutorials, white papers, webinars, and more.



This application guide was written in collaboration with Stephan Kreimer.

Stephan Kreimer is a master dental technician (MDT) specializing in digital workflow solutions and is a consultant/ beta tester for several dental companies. He owns and operates Kreimer Dentallabor GmbH & Co. KG, a dental laboratory in Warendorf, Germany.

To contact him, email [stephan@kreimer-dentallabor.de](mailto:stephan@kreimer-dentallabor.de) or visit <https://kreimer-dentallabor.de>





## Contact Sales

Contact Formlabs to learn how desktop stereolithography (SLA) 3D printing can simplify dental workflows and request a free sample part for your specific application.

Request a Free Sample Part

---

**North America Sales Inquiries**  
sales@formlabs.com  
617-702-8476  
[formlabs.com](https://formlabs.com)

**Europe Sales Inquiries**  
eu-sales@formlabs.com  
+44 330 027 0040  
[formlabs.com](https://formlabs.com)

**International Sales Inquiries**  
Find a reseller in your region:  
[formlabs.com/find-a-reseller](https://formlabs.com/find-a-reseller)